

ALBAcyte® REAGENT RED BLOOD CELLS FOR USE IN IDENTIFICATION OF UNEXPECTED ANTIBODIES

Expanded Rh Negative Antibody Screen - REF Z464U



Patient/Sample Details				
Name:	DOB:	Sample Identification:	ABO Rh:	
Diagnosis/History:	Hospital:	DAT:	Anti-IgG:	Anti-C3

Lot No:	V241576	
Expiry Date:	2022.01.03	U.S. License 1807

Cell #	Rh-hr	Donor	Rh-hr								Kell					Duffy		Kidd		Lewis		MNS				P	Lutheran		Additional Antigens			TEST RESULTS				Cell #			
			D	C	E	c	e	f	*V	C ^w	K	k	Kp ^a	Kp ^b	*Js ^a	*Js ^b	Fy ^a	Fy ^b	Jk ^a	Jk ^b	Le ^a	Le ^b	M	N	S	s	P1	Lu ^a	Lu ^b	Xg ^a	Wr ^a	Special Types							
1	r'r	4471906	0	+	0	+	+	+	0	0	+	+	0	+	0	+	0	+	+	0	0	0	+	+	+	+	+	0	+	+	0								1
2	r'r	1015702	0	0	+	+	+	+	NT	0	0	+	0	+	0	+	+	0	0	+	+	0	+	+	0	+	0	0	+	+	0								2
3	rr	1049850	0	0	0	+	+	+	NT	0	0	+	0	+	0	+	+	0	0	+	+	+	+	0	+	0	+	*0	0								3		
4	R ₀ R ₀	52161115	+	0	0	+	+	+	NT	0	0	+	0	+	0	+	0	0	+	+	0	+	0	+	0	+	+	0	+	*+	0								4
Patient Cells																																							

Notes:

- All cells are DAT negative.
- NT = Not tested.
- The f antigen status has been determined presumptively.
- Cell #4 is designated as R₀R₀ based on the statistical probability for the source donor population but a genotype of R₀r cannot be excluded.

* Indicates those antigens whose presence or absence may have been determined using only a single example of a specific antibody.

Conclusions/Further Testing Required:	
Signature/Initials:	Date: