

ALBAcyte® REAGENT RED BLOOD CELLS FOR USE IN IDENTIFICATION OF UNEXPECTED ANTIBODIES



Expanded Rh Negative Antibody Screen - REF Z464U

| Patient/Sample Details | | | | |
|------------------------|-----------|------------------------|-----------|---------|
| Name: | DOB: | Sample Identification: | ABO Rh: | |
| Diagnosis/History: | Hospital: | DAT: | Anti-IgG: | Anti-C3 |

| | | |
|---------------------|-------------------|--------------------------|
| Lot No: | V252819 | |
| Expiry Date: | 2022.11.07 | U.S. License 1807 |

| Cell # | Rh-hr | Donor | Rh-hr | | | | | | | Kell | | | | | Duffy | | Kidd | | Lewis | | MNS | | | | P | Lutheran | | Additional Antigens | | | TEST RESULTS | | | | Cell # | | | | | |
|---------------|-------------------------------|---------------|-------|---|---|---|---|---|----|----------------|---|---|-----------------|-----------------|------------------|------------------|-----------------|-----------------|-----------------|-----------------|-----------------|-----------------|---|---|---|----------|----|---------------------|-----------------|-----------------|-----------------|---------------|--|--|--------|--|--|--|--|---|
| | | | D | C | E | c | e | f | *V | C ^w | K | k | Kp ^a | Kp ^b | *Js ^a | *Js ^b | Fy ^a | Fy ^b | Jk ^a | Jk ^b | Le ^a | Le ^b | M | N | S | s | P1 | Lu ^a | Lu ^b | Xg ^a | Wr ^a | Special Types | | | | | | | | |
| 1 | r'r | 1095603 | 0 | + | 0 | + | + | + | NT | 0 | 0 | + | 0 | + | 0 | + | + | 0 | + | 0 | 0 | + | 0 | + | 0 | + | + | 0 | + | 0 | 0 | | | | | | | | | 1 |
| 2 | r''r | 3300886 | 0 | 0 | + | + | + | + | 0 | 0 | 0 | + | + | + | 0 | + | + | 0 | 0 | + | + | 0 | + | 0 | 0 | + | 0 | 0 | + | + | 0 | | | | | | | | | 2 |
| 3 | rr | 2768020835473 | 0 | 0 | 0 | + | + | + | NT | 0 | + | + | 0 | + | 0 | + | 0 | + | 0 | + | 0 | + | + | 0 | + | 0 | + | 0 | + | *+ | *0 | | | | | | | | | 3 |
| 4 | R ₀ R ₀ | 13578117 | + | 0 | 0 | + | + | + | NT | 0 | 0 | + | 0 | + | 0 | + | 0 | 0 | + | 0 | 0 | + | + | + | 0 | + | + | 0 | + | *0 | 0 | | | | | | | | | 4 |
| Patient Cells | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |

Notes:

1. All cells are DAT negative.
 2. NT = Not tested.
 3. The f antigen status has been determined presumptively.
 4. Cell #4 is designated as R₀R₀ based on the statistical probability for the source donor population but a genotype of R₀r cannot be excluded.
- * Indicates those antigens whose presence or absence may have been determined using only a single example of a specific antibody.

| |
|--|
| Conclusions/Further Testing Required: |
| Signature/Initials: Date: |