

ALBAcyte® REAGENT RED BLOOD CELLS FOR USE IN IDENTIFICATION OF UNEXPECTED ANTIBODIES

REF Z471U & Z472U



Patient/Sample Details				
Name:	D.O.B:	Sample Identification:	ABO Rh:	
Diagnosis History:	Hospital:	DAT:	Anti-IgG:	Anti-C3:

Lot No:	V252849 & V252852
Expiry Date:	2022.11.07
U.S License	1807

Cell #	Rh-hr	Donor	Rh-hr							Kell					Duffy		Kidd		Lewis		MNS				P	Lutheran		Additional Antigens			TEST RESULTS				Cell #					
			D	C	E	c	e	f	*V	C ^w	K	k	Kp ^a	Kp ^b	*Js ^a	*Js ^b	Fy ^a	Fy ^b	JK ^a	JK ^b	Le ^a	Le ^b	M	N	S	s	P1	Lu ^a	Lu ^b	Xg ^a	Wr ^a	Special Types								
1	R ₁ ^w R ₁	5908240825012	+	+	0	0	+	0	NT	+	0	+	0	+	0	+	+	0	+	0	+	+	+	+	+	+	0	+	+	0									1	
2	R ₁ R ₁	3376706	+	+	0	0	+	0	NT	0	+	+	0	+	0	+	+	0	+	0	+	0	+	0	+	0	+	+	*0										2	
3	R ₂ R ₂	15236896	+	0	+	+	0	0	NT	0	0	+	0	+	0	+	+	+	0	0	+	0	+	+	+	0	0	+	*+	*0										3
4	r'r	1231207	0	+	0	+	+	+	NT	0	+	+	0	+	0	+	+	0	+	+	0	+	+	+	+	0	0	+	0	0										4
5	r'r	6610682	0	0	+	+	+	+	NT	0	0	+	0	+	0	+	+	0	+	+	0	+	+	+	+	0	+	+	*+	*0										5
6	R ₀ r	1413-573-5	+	0	0	+	+	+	NT	0	0	+	0	+	0	+	+	0	+	0	+	+	0	+	+	0	+	+	*0	*0										6
7	rr	1359-893-3	0	0	0	+	+	+	NT	0	+	+	0	+	0	+	+	+	0	+	+	+	+	0	+	0	+	+	*+	*0										7
8	rr	3306410	0	0	0	+	+	+	0	0	0	+	0	+	0	+	+	0	+	0	+	+	0	+	+	0	+	+	0	0										8
9	rr	3336181	0	0	0	+	+	+	0	0	0	+	+	0	+	0	+	+	0	+	0	+	+	0	+	0	+	+	0	0										9
10	rr	11086972	0	0	0	+	+	+	0	0	0	+	0	+	0	+	+	0	+	0	+	+	0	+	+	0	+	+	0	0										10
Patient Cells																																								

Notes:

- All cells are DAT negative.
 - M, N, S, s, Fy^a and Fy^b antigens are destroyed or depressed following enzyme treatment.
 - 's' = strong reactor, 'w' = weak reactor, NT = not tested
 - The f antigen status may have been determined presumptively based on the Rh-hr phenotype.
- * Indicates those antigens whose presence or absence may have been determined using only a single example of a specific antibody.

Conclusions/Further Testing Required:	
Signature/Initials:	Date: