

ALBAcyte®REAGENT RED BLOOD CELLS FOR IDENTIFICATION OF UNEXPECTED ANTIBODIES
Antibody Identification (16-Cell) - REF Z473U



Patient/Sample Details				
Name:	D.O.B:	Sample Identification:	ABO Rh:	
Diagnosis History:	Hospital:	DAT:	Anti-IgG:	Anti-C3:

Lot No:	V252854
Expiry Date:	2022.11.07 U.S License 1807

Cell #	Rh-hr	Donor	Rh-hr							Kell				Duffy		Kidd		Lewis		MNS				P	Lutheran		Additional Antigens			TEST RESULTS				Cell #		
			D	C	E	c	e	f	*V	C ^w	K	k	Kp ^a	Kp ^b	*Js ^a	*Js ^b	Fy ^a	Fy ^b	JK ^a	JK ^b	Le ^a	Le ^b	M	N	S	s	P1	Lu ^a	Lu ^b	Xg ^a	Wr ^a	Special Types				
1	R ₁ ^w R ₁	5908240825012	+	+	0	0	+	0	NT	+	0	+	0	+	0	+	+	+	0	+	+	+	+	+	0	+	+	0						1		
2	R ₁ R ₁	3376706	+	+	0	0	+	0	NT	0	+	+	0	+	0	+	0	+	+	0	+	0	+	0	+	0	+	+	*0						2	
3	R ₂ R ₂	15236896	+	0	+	+	0	0	NT	0	0	+	0	+	0	+	+	+	0	0	+	0	+	+	0	+	+	*+	*0						3	
4	r'r	1231207	0	+	0	+	+	+	NT	0	+	+	0	+	0	+	+	0	+	+	0	+	+	0	+	0	+	0	0						4	
5	r''r	6610682	0	0	+	+	+	+	NT	0	0	+	0	+	0	+	0	+	+	0	+	0	+	+	0	+	+	*+	*0						5	
6	R ₀ r	1413-573-5	+	0	0	+	+	+	NT	0	0	+	0	+	0	+	0	0	+	+	0	0	+	+	0	+	0	+	*0	*0						6
7	rr	1359-893-3	0	0	0	+	+	+	NT	0	+	+	0	+	0	+	+	0	+	+	+	+	0	+	0	+	+	*+	*0						7	
8	rr	3306410	0	0	0	+	+	+	0	0	0	+	0	+	0	+	+	+	0	+	+	0	+	0	+	+	+	0	0						8	
9	rr	3336181	0	0	0	+	+	+	0	0	0	+	+	0	+	+	0	+	+	0	+	0	+	0	+	0	+	+	0						9	
10	rr	11086972	0	0	0	+	+	+	0	0	0	+	0	+	0	+	+	0	+	+	+	+	0	+	0	+	+	0							10	
11	R ₁ R ₂	628325788	+	+	+	0	+	0	NT	0	0	+	0	+	0	+	+	0	+	0	+	0	+	+	0	+	0	+	+	0						11
12	R ₁ R ₁	4247367	+	+	0	0	+	0	NT	0	0	+	0	+	0	+	+	+	0	+	+	0	+	+	0	+	0	+	*0	*0						12
13	R ₁ r	5747950711	+	+	0	+	+	+	NT	0	+	0	0	+	0	+	0	+	+	+	0	+	+	+	+	+	+	*+	*0							13
14	R ₂ R ₂	56107512	+	0	+	+	0	0	NT	0	+	+	0	+	0	+	+	+	+	0	+	+	+	0	+	0	+	*+	0							14
15	R ₂ R ₂	54916962	+	+	+	+	0	0	NT	0	0	+	0	+	0	+	0	+	+	+	0	+	0	+	0	+	0	+	*+	0						15
16	rr	3908516	0	0	0	+	+	+	0	0	0	+	0	+	0	+	0	0	+	0	0	+	+	+	+	+	0	+	+	0						16
Patients cells																																				

Notes:
 1. All cells are DAT negative.
 2. 's' = strong reactor, 'w' = weak reactor, NT = not tested
 3. The f antigen status has been determined presumptively
 4. Cell #6 is designated as R₀r but the phenotyping test results cannot exclude the possibility of the red blood cell genotype being R₀R₀.

* Indicates those antigens whose presence or absence may have been determined using only a single example of a specific antibody.
 Quotient, 301 South State Street, S-204, Newtown, PA 18940, USA
 Product Technical Support Tel: 1-888-228-1990 Web: www.quotientbd.com

Conclusions/Further Testing Required:	
Signature/Initials:	Date: