

ALBAcyte® REAGENT RED BLOOD CELLS FOR USE IN IDENTIFICATION OF UNEXPECTED ANTIBODIES

REF Z471U & Z472U



Patient/Sample Details				
Name:	D.O.B:	Sample Identification:	ABO Rh:	
Diagnosis History:	Hospital:	DAT:	Anti-IgG:	Anti-C3:

Lot No:	V264001 & V264003
Expiry Date:	2023.10.09 U.S License 1807

Cell #	Rh-hr	Donor	Rh-hr					Kell				Duffy		Kidd		Lewis		MNS				P	Lutheran		Additional Antigens			TEST RESULTS				Cell #																	
			D	C	E	c	e	f	*V	C ^w	K	k	Kp ^a	Kp ^b	*Js ^a	*Js ^b	Fy ^a	Fy ^b	Jk ^a	Jk ^b	Le ^a	Le ^b	M	N	S	s	P1	Lu ^a	Lu ^b	Xg ^a	Wr ^a		Special Types																
1	R ₁ ^w R ₁	7312122105015	+	+	0	0	+	0	NT	+	0	+	0	+	+	+	0	+	+	+	+	+	+	0	0	+	*+	0							1														
2	R ₁ R ₁	4143835	+	+	0	0	+	0	0	0	+	+	0	+	0	+	+	0	+	0	+	+	0	+	0	+	0	0									2												
3	R ₂ R ₂	56513040	+	0	+	+	0	0	NT	0	0	+	0	+	+	+	0	+	+	0	+	+	0	+	0	+	*+	0									3												
4	r'r	1231207	0	+	0	+	+	+	NT	0	+	+	0	+	0	+	+	0	+	+	0	+	+	+	0	+	0	0									4												
5	r'r	772531	0	0	+	+	+	+	NT	0	0	+	0	+	+	+	+	0	+	0	0	0	+	0	+	+	0	+	*+	0							5												
6	R ₀ r	1207-926-5	+	0	0	+	+	+	NT	0	0	+	0	+	NT	+	0	0	+	0	0	+	+	0	+	+	0	+	*0	0								6											
7	rr	1392-462-6	0	0	0	+	+	+	NT	0	+	+	0	+	0	+	+	+	+	0	+	0	+	0	+	+	+	*0	0									7											
8	rr	943993	0	0	0	+	+	+	NT	0	0	+	0	+	0	+	0	+	+	+	+	0	+	+	0	+	0	0										8											
9	rr	1300117	0	0	0	+	+	+	NT	0	0	+	0	+	0	+	0	+	+	+	0	+	+	0	+	+	0	0										9											
10	rr	4136	0	0	0	+	+	+	NT	0	0	+	+	0	+	+	0	+	+	+	+	+	+	+	+	0	+	+	0										10										
Patient Cells																																																	

- Notes:**
1. All cells are DAT negative.
 2. M, N, S, s, Fy^a and Fy^b antigens are destroyed or depressed following enzyme treatment.
 3. 's' = strong reactor, 'w' = weak reactor, NT = not tested
 4. The f antigen status may have been determined presumptively based on the Rh-hr phenotype.
- * Indicates those antigens whose presence or absence may have been determined using only a single example of a specific antibody.

Conclusions/Further Testing Required:		
<table style="width:100%; border: none;"> <tr> <td style="width:50%; border: none;">Signature/Initials:</td> <td style="width:50%; border: none;">Date:</td> </tr> </table>	Signature/Initials:	Date:
Signature/Initials:	Date:	